


### **Titrating the EMA appliance**

The **EMA** oral appliance from EMA-Sleep, Inc. increases airway space by advancing the mandible using interchangeable elastic straps. EMA elastic straps come in **9 different lengths** (see figure A), and **4 different strengths** (see figure B). The shorter the EMA elastic strap, the farther the mandible is advanced (see figure C). **36 different straps** allow for the most effective repositioning with maximum comfort.

### 9 different lengths allow for 1 mmFigure Aadvancement of the mandible

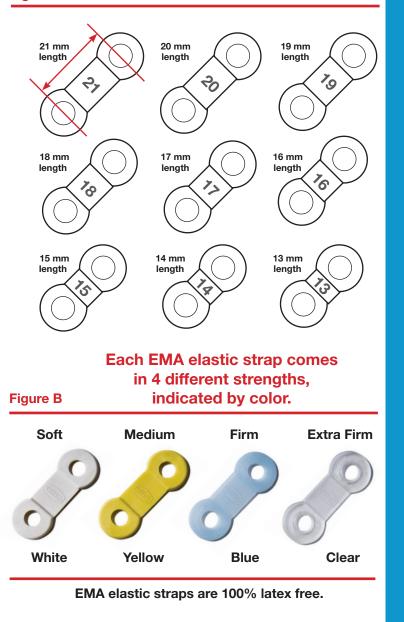




## The shorter the EMA elastic strap, the farther the mandible is advanced.

Figure C

EMA buttons are 27 mm apart\*

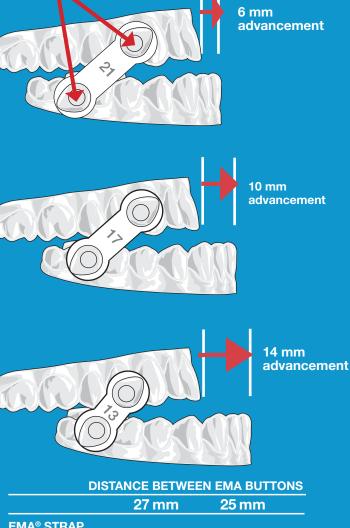

AMOUNT OF A	DVANCEMENT
6 mm	4 mm
7 mm	5 mm
8 mm	6 mm
9 mm	7 mm
10 mm	8 mm
11 mm	9 mm
12 mm	10 mm
13 mm	
14 mm	
	6 mm 7 mm 8 mm 9 mm 10 mm 11 mm 12 mm 13 mm

\*EMA-Sleep, Inc. recommends a distance between the buttons of 27 mm in the centric position. If the patient's range of mandibular advancement is limited to 9 mm or fewer, a distance of 25 mm distance is recommended.




# SINPLICETY EVOLVED

#### **PROVEN SLEEP TECHNOLOGY**



Patented, custom design resulting in comfort and effectiveness

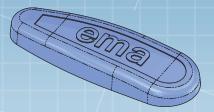

Minimal profile and anatomically correct occlusal pad to maximize oral cavity volume



9 different lengths allow for 1 mm advancement of the mandible

- Durable, biocompatible, and medical grade nylon
- Efficient manufacturing process resulting in a high value, quality product