


Titrating the EMA appliance

The **EMA** oral appliance from EMA-Sleep, Inc. increases airway space by advancing the mandible using interchangeable elastic straps. EMA elastic straps come in 9 different lengths (see figure A), and 4 different strengths (see figure B). The shorter the EMA elastic strap, the farther the mandible is advanced (see figure C). 36 different straps allow for the most effective repositioning with maximum comfort.

9 different lengths allow for 1 mm advancement of the mandible **Figure A**

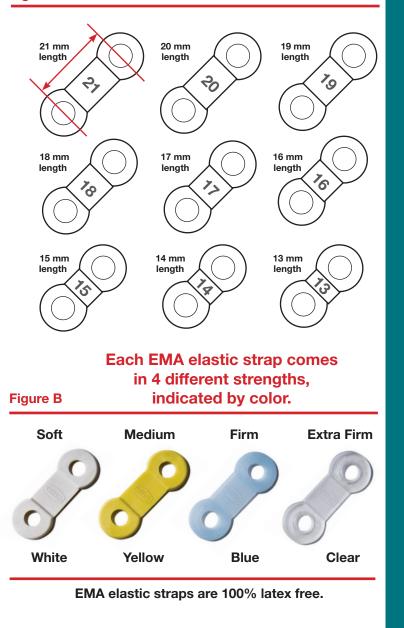




The shorter the EMA elastic strap, the farther the mandible is advanced.

Figure C

EMA buttons are 27 mm apart*

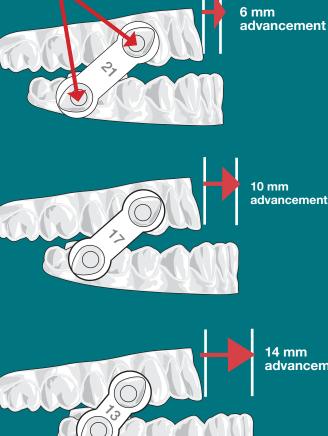

advancement

	DISTANCE BETWEEN EMA BUTTONS		
	27 mm	25 mm	
EMA® STRAP			
LENGTH	AMOUNT OF A	AMOUNT OF ADVANCEMENT	
21 mm	6 mm	4 mm	
20 mm	7 mm	5 mm	
19 mm	8 mm	6 mm	
18 mm	9 mm	7 mm	
17 mm	10 mm	8 mm	
16 mm	11 mm	9 mm	
15 mm	12 mm	10 mm	
14 mm	13 mm		
13 mm	14 mm		

*EMA-Sleep, Inc. recommends a distance between the buttons of 27 mm in the centric position. If the patient's range of mandibular advancement is limited to 9 mm or fewer, a distance of 25 mm distance is recommended.




SINPLICITY WITH DURABILITY

PROVEN SLEEP TECHNOLOGY



Patented, custom design resulting in comfort and effectiveness

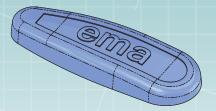

Minimal profile and anatomically correct occlusal pad to maximize oral cavity volume



9 different lengths allow for 1 mm advancement of the mandible

- Durable, biocompatible, and proprietary EMA CAD Thermoplastic
- Advanced Milling process resulting in a durable and comfortable product